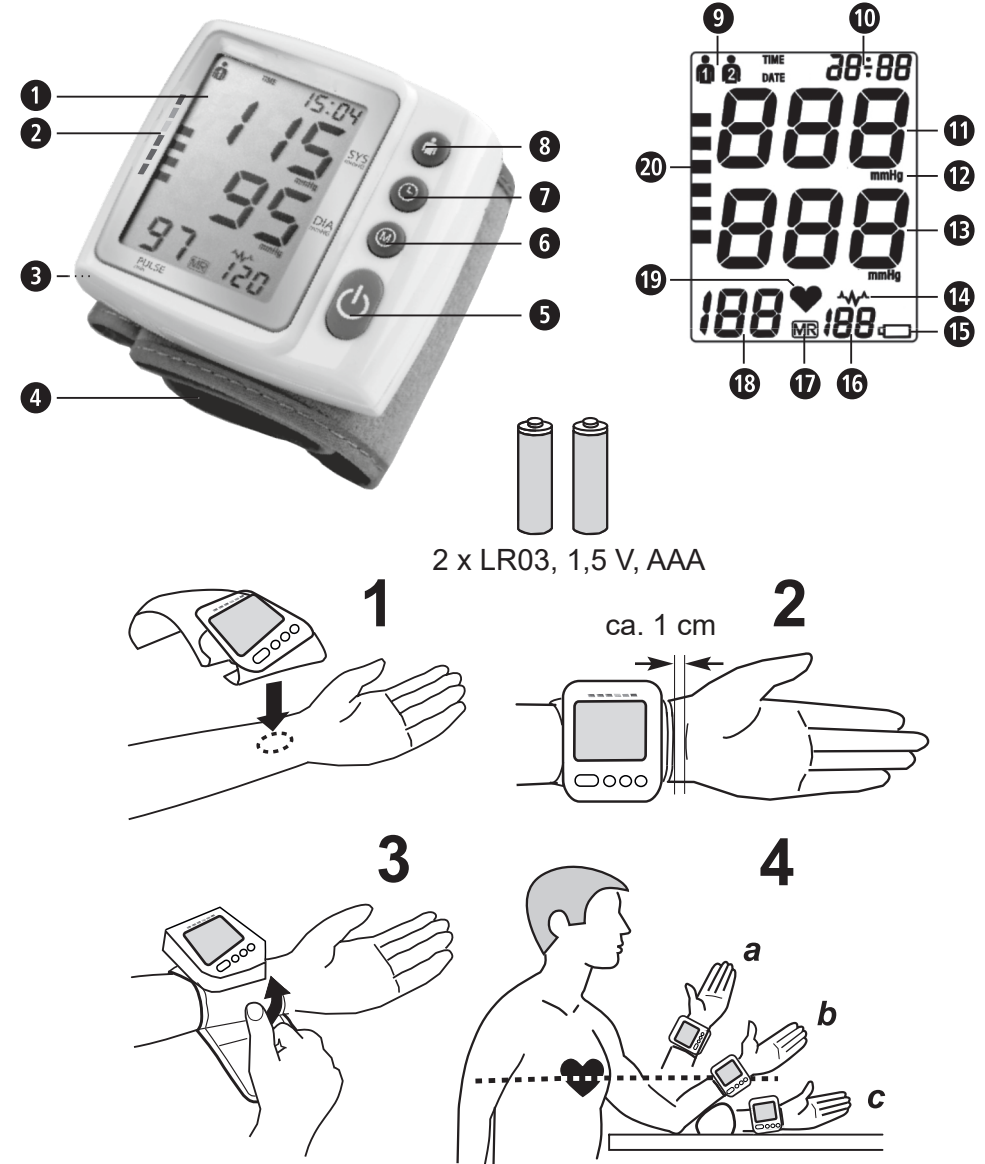
medisana®

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
medisana®

GB Instruction manual
Blood Pressure Monitor BW 315

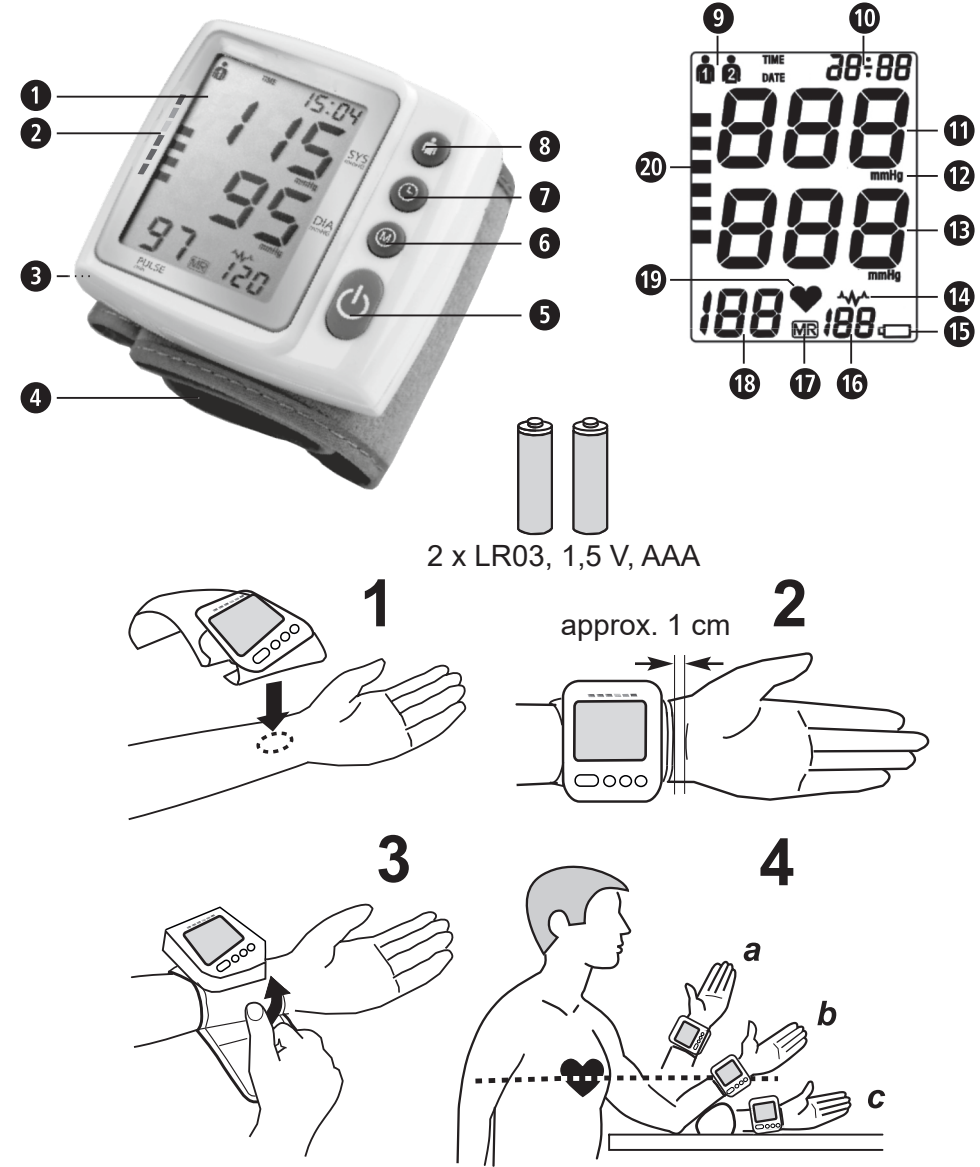Device and controls

Intended use

- The unit is intended only for private use in the home.
- The blood pressure monitor is for use by medical professionals, operators, or at home. It is a non-invasive blood pressure measurement system intended to measure the diastolic and systolic blood pressures and pulse rate of an adult individual by using a non-invasive technique in which an inflatable cuff is wrapped around the wrist.

Indications

- This device is used to monitor adult blood pressure and pulse.

Contraindications

- The device is not suitable for measuring the blood pressure of children. Ask your doctor before using it on older children.
- This item is not suitable for persons who suffer from a strong arrhythmia.
- Do not use this device with a defibrillator.
- Do not use this device during an MRI examination.
- Do not use the device in a flammable environment (e.g. an oxygen-enriched environment).
- Do not immerse the device in water or other liquids. Do not use acetone or other volatile solutions to clean the device.
- If you have had a mastectomy, please consult your doctor before using this monitor.
- Do not use the monitor in a moving vehicle, such as a car or an airplane.
- Avoid bathing, drinking alcohol or caffeine, smoking, exercising, and eating at least 30 minutes before the measurement.
- Do not use this device with a defibrillator.
- Do not use this device during an MRI examination.
- Do not use the device in a combustible environment (i.e., oxygen-enriched environment).
- Never submerge the device in water or other liquids. Do not clean the device with acetone or other volatile solutions.
- If you have had a mastectomy or lymph node removal, talk to your doctor before using this monitoring device.
- DO NOT use this monitor in a moving vehicle such as in a car or on an aircraft.
- Avoid bathing, drinking alcohol or caffeine, smoking, exercising and eating for at least 30 minutes before taking a measurement.

Intended users

- Medical staff or patients who can use the product according to the instruction manual.

Intended patient population

- This device is for adult patients.
- Consult with your physician before using this monitor if you have common arrhythmias such as atrial or ventricular premature beats or atrial fibrillation; arterial sclerosis; poor perfusion; diabetes; pregnancy; pre-eclampsia or renal disease. NOTE that any of these conditions in addition to patient motion, trembling, or shivering may affect the measurement reading.

Intended Use Environment

- Operation conditions 5–40°C, 15%–85% RH (non-condensing), 700 hPa–1060 hPa.

Clinical Benefits to be Expected:

- The medisana Blood Pressure Monitor BW 315 offers blood pressure measurements with an accuracy that meets regulatory requirements for users who want to measure blood pressure values.

Explanation of symbols

IMPORTANT
Refer to the instruction manual/booklet!
Non-observance of these instructions can result in serious injury or damage to the device.

WARNING – General Warning Sign
These warning notes must be observed to prevent any injury to the user.

CAUTION
These notes must be observed to prevent any damage to the device.

NOTE
These instructions provide you with useful additional information regarding installation or operation.

Consult instructions for use or consult electronic instructions for use. This indicates the need for the user to consult the instructions for use

General prohibition sign

Warning: Not suitable for children under 3 years!

IP22
Ingress protection degree provided by IEC 60529

Disposal only according to the “WEEE” Waste Electrical and Electronic Equipment EC Directive.
– Electrical and electronic equipment mark: Reduce electronic and electrical waste as unsorted waste and collect it separately

Recycling symbols/codes: These are used to provide information about the material and its proper use and recycling.

CE 0197
CE marking of conformity. This product complies with the European Medical Device Regulation (EU) 2017/745.

REF
Catalogue Number

#
Model Number

SN
Serial Number

MD
This indicates that the item is a Medical device

EU REP
Authorized representative in the European Community / European Union

UDI
Unique Device Identifier

Fragile, handle with care

Paper recycling

Importer

Manufacturer
– This indicates the manufacturer of the medical device

LOT
Batch code
– This indicates the manufacturer's batch code so that the batch or lot can be identified

Applied part – Type BF degree of protection against electric shock (leakage current)

Humidity limitation
– This indicates the range of humidity the device can be safely exposed to

Ambient pressure limitation

Temperature limit
– This indicates the limit temperatures the device can be safely exposed to

Maximum number of identical transport packages/items that may be stacked on the bottom package, where “6” is the limiting number.

This is the correct upright position of the distribution packages for transport and/or storage.

Keep dry

Date of manufacture

Direct voltage / Direct current

GB

**IMPORTANT INFORMATION!
KEEP IN A SAFE PLACE!**

Please read this instruction manual carefully, in particular the safety instructions, before using the device. Keep the instruction manual in a safe place for later reference. If you pass the device on to a third party, this instruction manual must remain with the device.

-
- The Wrist Blood Pressure Monitor (integrated time/date display) is a fully automatic digital blood pressure measuring device that can measure systolic and diastolic blood pressure very quickly and reliably.
 - Please read this instruction manual carefully before use. And keep it safe. For further questions about blood pressure and its measurement, please contact your doctor.
 - This device is a non-invasive blood pressure measurement system intended to measure the diastolic and systolic blood pressures and pulse rate of an adult individual by using a non-invasive technique in which an inflatable cuff is wrapped around the wrist.
 - For special populations, please consult your doctor before using this device.
 - Read the Important Safety Information in this instruction manual before using this monitor. Follow this instruction manual thoroughly for your safety. Keep for future reference. For specific information about your own blood pressure, CONSULT WITH YOUR PHYSICIAN.
 - Before using the device, please ensure that you have read this manual thoroughly and fully understand corresponding precautions and risks.
 - Do not drop this device or subject it to strong impact.
 - Do not place this device in pressure vessels or gas sterilization device.
 - Do not dismantle the device, as this could cause damage or malfunctions or impede the operation of the device.
 - This device is not intended for use by people (including children) with restricted physical, sensory or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who has responsibility for their safety or they receive instructions from this person on how to use the device. Children should be supervised around the device to ensure they do not play with it.
 - Do not store the device in the following locations: locations in which the device is exposed to direct sunlight, high temperatures or levels of moisture, or heavy contamination; locations near to sources of water or fire; or locations that are subject to strong electromagnetic influences.
 - Blood pressure measurements, such as those taken with this device, cannot identify all diseases. Regardless of the measurement taken using this device, you should consult your doctor immediately if you experience symptoms that could indicate acute disease.
 - Do not self-diagnose or self-medicate on the basis of this device without consulting your doctor. This is exclusively the responsibility of your doctor. In particular, do not start taking any new medication or change the type and/or dosage of any existing medication without prior approval.
 - Clean the device and wrist cuff with a dry, soft cloth or a cloth dampened with water and a neutral detergent. Never use alcohol, benzene, thinner or other harsh chemicals to clean the device or cuff.
 - To measure blood pressure, the wrist must be squeezed by the wrist cuff hard enough to temporarily stop blood flow through the artery. This may cause pain, numbness or a temporary red mark to the wrist. This condition will appear especially when measurement is repeated successively.
 - Any pain, numbness, or red marks will disappear with time.
 - People who have a severe circulatory deficit in the wrist must consult a doctor before using the device, to avoid medical problems.
 - Do not repair or maintain the equipment during use to avoid incorrect operation of the equipment and deviation or error of the measured value.
 - Do not measure too frequently, which may cause pain and numbness to the user's wrist due to obstructed blood flow.
 - Do not use a wrist cuff on a wrist with a wound as it may cause further injury.
 - When the cuff is applied to any limb and pressure is applied, the measurement can be stopped if the pressure temporarily interferes with the flow of blood and may cause numbness in the wrist.

- By observing that there are no obvious symptoms of discomfort in the limb, the operation of the blood pressure monitor will not cause long-term damage to the patient's blood circulation.
 - Stop using this monitor and consult with your physician if you experience skin irritation or discomfort.
 - Remove the wrist cuff if it does not start deflating during a measurement.
 - DO NOT use this monitor for any purpose other than measuring blood pressure.
 - During measurement, make sure that no mobile device or any other electrical device that emit electromagnetic fields is within 30 cm of this monitor. This may result in incorrect operation of the monitor and/or cause an inaccurate reading.
 - DO NOT use this monitor with other medical electrical (ME) equipment simultaneously. This may result in incorrect operation and/or cause an inaccurate reading.
 - Rest for at least 5 minutes before taking a measurement.
 - Remove tight-fitting or thick clothing from your arm while taking a measurement.
 - Remain still and DO NOT talk while taking a measurement.
 - ONLY use the wrist cuff on persons whose wrist circumference is within the specified range of the cuff (see “Technical Specifications”).
 - Ensure that this monitor has acclimated to room temperature before taking a measurement. Taking a measurement after an extreme temperature change could lead to an inaccurate reading. Recommends waiting for approximately 2 hours for the monitor to warm up or cool down when the monitor is used in an environment within the temperature specified as operating conditions after it is stored either at the maximum or at the minimum storage temperature. For additional information on operating and storage/transport temperature, refer to section “Technical specifications”.
 - DO NOT use this monitor after the durable period has ended. Refer to section “Technical specifications”.
 - When wrapping the strap around the wrist, avoid squeezing or other external forces, and only use the wrist strap for the types of persons as specified in this manual.
 - ONLY use the approved wrist cuff for this monitor. Use of other wrist cuffs may result in incorrect readings.
 - Read and follow the section “Disposal” when disposing of the device and any used accessories or optional parts.
- BATTERY SAFETY INFORMATION**
- Do not disassemble batteries!
 - Clean the battery and device contacts if necessary before putting in the batteries!
 - Remove discharged batteries from the device immediately!
 - Increased risk of leakage, avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of this parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
 - If a battery has been swallowed seek medical attention immediately!
 - Replace all of the batteries simultaneously!
 - Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!
 - Insert the batteries correctly, observing the polarity!
 - Keep the battery compartment well sealed!
 - Remove the batteries from the device if it is not going to be used for an extended period!
 - Keep batteries out of the reach of infants, toddlers and children.
 - Do not attempt to recharge these batteries! There is a danger of explosion!
 - Do not short circuit! There is a danger of explosion!
 - Do not throw into a fire! There is a danger of explosion!
 - Keep unused batteries in their packaging away from metal objects in order to prevent short circuiting!
 - Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!
 - Use only 2 “AAA” alkaline batteries with this device. Rechargeable batteries cannot be used.
 - Do not use batteries after their expiration date.
 - Check the batteries regularly to ensure they are in good working condition.
 - Disposal of used batteries should be carried out in accordance with local regulations
 - The supplied batteries may have a shorter life span than new batteries.
 - Remember to have a record of your blood pressure and pulse readings for your physician. A single measurement does not provide an accurate indication of your true blood pressure.

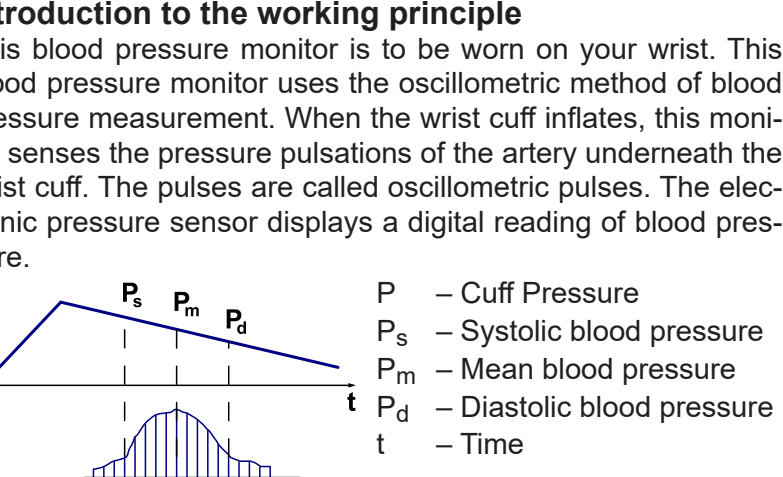
- Unit and LCD Display**
- 1 LCD display
 - 2 Blood Pressure Indicator (green - yellow - orange - red)
 - 3 Lid of the Battery Compartment
 - 4 Wrist cuff
 - 5 ⌂ button (ON/OFF)
 - 6 ⌂ button (Memory recall)
 - 7 ⌂ button (Clock/Date)
 - 8 * button (User Setting)
 - 9 User memory 1/2
 - 10 Display of Clock / Date
 - 11 Display of Systolic Pressure
 - 12 Unit of measure
 - 13 Display of Diastolic Pressure
 - 14 Irregular heartbeat symbol
 - 15 Change Battery symbol
 - 16 Memory location number
 - 17 Memory symbol
 - 18 Display of Pulse Rate
 - 19 Pulse symbol
 - 20 Blood Pressure Indicator

WARNING
Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

What is blood pressure?

Blood pressure is the pressure created in the blood vessels by each heart beat. When the heart contracts (= systole) and pumps blood into the arteries, it brings about a rise in pressure. The highest value is known as the systolic pressure and is the first value to be measured. When the cardiac muscle relaxes to take in fresh blood, the pressure in the arteries also falls. Once the arteries are relaxed, a second reading is taken – the diastolic pressure.

Both pressures, the Systolic and Diastolic, are necessary to enable a physician to evaluate the status of a patient's blood pressure.

The pressure sensor collects the pressure in the wrist cuff, converts it into digital signal and sends it to CPU. Through embedded software, the corresponding pressure points in the process of blocking arterial blood flow are identified. The diastolic blood pressure, systolic blood pressure and mean pressure of the body are obtained based on the accumulated software algorithm.

Blood pressure classification

The lines ② on the left-hand side of the display point at the range within which the measured blood pressure value lies. The value is either within the optimum, elevated or high range. The classification corresponds to the ranges printed in the table. These ranges are defined by international guidelines (2023 ESH).

WARNING
Blood pressure that is too low represents just as great a health risk as blood pressure that is too high! Fits of dizziness may lead to dangerous situations arising (e.g. on stairs or in traffic)!

Classification of Hypertension

The values in the following table are provided by the 2023 ESH Guidelines. The blood pressure category is defined by the highest level of blood pressure, whether systolic or diastolic. Isolated systolic or diastolic hypertension is graded 1, 2 or 3 according to systolic blood pressure and diastolic blood pressure values in the indicated ranges. The same classification is used for adolescents ≥16 years old.

Category	Systolic [mmHg]	Diastolic [mmHg]
Optimal	< 120	< 80
Normal	120 – 129	80 – 84
High-Normal	130 – 139	85 – 89
Grade 1 Hypertension	140 – 159	90 – 99
Grade 2 Hypertension	160 – 179	100 – 109
Grade 3 Hypertension	≥ 180	≥ 110
Isolated Systolic Hypertension	≥ 140	≤ 90
Isolated Diastolic Hypertension	≤ 140	≥90

Influencing and evaluating readings

- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. When using the unit regularly and recording the values for your doctor, you should visit the doctor from time to time to keep him updated.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least 5-10 minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

Starting up

Insert / replace batteries

You must insert the batteries provided before you can use your unit. The lid of the battery compartment  is located on the left hand side of the unit. Open it and insert the 2 AAA LR03 type 1.5 V batteries supplied. Ensure correct polarity when inserting the batteries (as marked inside the battery compartment). Close the battery compartment. Replace the batteries immediately, if the change battery symbol  appears on the display  or if nothing appears on the display after the unit has been switched on.

Setting date and clock

After insertion of the batteries you need to set the date and clock. To enter the setting mode, press and hold the  button  for approx. 3 seconds. Firstly, the user memory 1 or 2  is flashing. Now press the  button  again to adjust the correct year figure. Adjust to correct year by pressing  button  and press  button  to confirm the setting. The device now switches to the month setting mode. Proceed accordingly (in the same way as with the year adjustment) to adjust the correct figures for month, day, hour and minute. Afterwards, the setting procedure is complete. Shortly, the current date appears and then the clock appears. If you want to look at the date later on, press the  button  shortly. If the batteries are changed, the set information is lost and needs to be entered again.

Setting up the user memory

The medisana wrist blood pressure monitor BW 315 gives you the opportunity to assign readings to one of two user memories, each of which provides 120 memory slots. Press the  button  to select the user memory  or . Your selection will be saved by the device and will be considered in next blood pressure measurement respectively when the measurement result is saved. Your selection is valid until you make another selection.

Fitting the cuff

- Attach the clean cuff to the left wrist which is free from clothing with the palm of the hand facing upwards (fig.1).
- There should be a gap of approx. 1 cm between the cuff and the palm of the hand (fig.2).
- Press the Velcro tape firmly around the wrist ensuring that it is not so tight that it could falsify the measured result (fig.3).

Correct position for taking readings

- Carry out the measurement whilst seated.
- Relax your arm and place it comfortably, on a table for instance.
- Raise the wrist so that the cuff of the monitor is at heart level (fig. 4, position a = too high, b = correct, c = too low).
- Keep still during the measurement: Do not move and do not talk as this may alter the measurements.

Taking a blood pressure measurement

After the cuff has been appropriately positioned,the measurement can begin:

- Press the  button  to start the measurement.
- The device automatically inflates the cuff slowly in order to measure your blood pressure. The rising pressure in the cuff is shown on the display.
- The unit inflates the cuff until sufficient pressure has built up for a measurement. Then the unit slowly releases air from the cuff and carries out the measurement. When the device detects the signal, the heart symbol   starts to flash on the display.
- When the measurement is finished, the cuff is deflated completely. The systolic and diastolic blood pressure and the pulse value appear on the display . The blood pressure indicator  is shown next to the relevant coloured bar depending on the blood pressure classification. If the unit has detected an irregular heartbeat, the symbol   is also shown.

WARNING
Do not take any therapeutic measures on the basis of a self measurement. Never alter the dose of a medicine prescribed by a doctor.

- The readings are automatically saved in the selected memory ( or ). Up to 120 measured values with date and time can be stored in each memory.
- The measurement readings remain on the display. The unit switches off automatically after approx. 3 minutes if no buttons are pressed. The unit may also be switched off using the  button .

Discontinuing a measurement

If it is necessary to interrupt a blood pressure measurement for any reason (e.g. the patient feels unwell) the  button  can be pressed at any time. The device immediately decrease the cuff pressure automatically.

Display stored values

This unit features 2 separate memories, each with a capacity of 120 memory slots. Results are automatically stored in the memory selected by the user. Press the  button  to select the user ( or ). Now press the  button  and the average values of the last 3 measurements appear on the display. Press the  button  again and the latest stored measurement value will be shown, together with the memory symbol „MR“  and the memory location place „1“ . Repeatedly pressing the  button  displays the respective values measured previously (MR2, MR3 and so on). If you have reached the final entry and do not press a button, the unit automatically switches itself off after approximately 3 min. Memory recall mode can be exited at any time by pressing the  button . The oldest value is deleted, if 120 measured values have been stored in the memory and a new value is saved.

Delete saved values

If you are sure that you want to permanently remove all stored memories of a user, press and hold the  button  for approx. 5 seconds, until CL appears in the display. Release the button. Now press the  button  again to confirm the deletion.

Error indicators

The following error codes will appear on the display when a measurement is abnormal:

Display	Cause
ERR1	No pulse was detected.
ERR2	Movement or talking during measurement.
ERR3	Inflation process lasts too long. The cuff probably has not been positioned properly.
ERR5	Systolic and diastolic pressure are too far apart from each other. Repeat the measurement. If you constantly receive unusual measurement results, please contact your doctor.
ERR8	Pressure in cuff is over 290mmHg

Troubleshooting

Problem	Cause and solutions
No display although the device is switched on and batteries are inserted.	Maybe the batteries are weak. Insert new batteries and pay attention to correct polarity. Check, if the device is working well. In the case of doubt, contact the service centre.
Measurement cannot be proceeded or measurement failed.	Put on the cuff correctly. Repeat the measurement after relaxing 30 minutes. Do not speak or move during the measurement.
Each measurement result strongly deviates from the one before.	Repeat the measurement in correct way after relaxing 30 minutes. Pay attention to all the instructions in this manual and to the common factors of wrong measurements. Slightly deviating measurement results are quite normal, as the blood pressure is constantly changing.
The measured values are remarkably different from values measured in a doctors surgery.	Store the measured results and consult your doctor. Please note: Individuals visiting their doctor frequently experience anxiety which can result in a higher reading at the doctor than obtained at home under resting conditions.

Please contact the manufacturer resp. the service centre if you can't solve the problem. Do not disassemble the unit by yourself.

Common factors of wrong measurements

- Please keep quiet for 5-10 minutes and avoid eating, don't drink alcohol or caffeine, no smoking, no exercising, no bathing before taking a measurement. All these factors will influence the measurement result.
- Remove watches or other spruce from your wrist before measurement.
- Always measure on the same wrist (normally left).
- Measurements should be taken regularly at the same time of each day, as the blood pressure varies even during the day.
- Any effort to support the arm during measurement may increase the measured blood pressure.
- Make sure, you are in a comfortable, relaxed position and do not move or constrict your muscles during measurement. Use a cushion to support your arm if necessary.
- Extreme temperatures, humidity or altitudes may impair the performance of the device.
- If the wrist is not at heart level, a false reading may be obtained.
- A loose or open cuff causes false readings.
- With repeated measurements, blood accumulates in the wrist, what may lead to false readings.
- Consecutive blood pressure measurements should be carried out after a 3-minute pause or after the arm has been held up in order to allow the accumulated blood to flow normally.

Maintenance, Storage and Cleaning

Remove the batteries before cleaning the unit. Clean the unit and the cuff using a soft cloth lightly moistened with a mild soapy solution. Never use abrasive cleaning agents, alcohol, naphtha, thinner or gasoline etc. Never immerse the unit or any component in water. Be cautious not to get any moisture in the main unit. Do not use the unit again until it is completely dry. Only inflate the cuff when it is in position around the wrist. Do not expose the unit to direct sunlight; protect it against dirt and moisture. Do not subject the unit to extreme hot or cold temperature. Keep the unit in the storage box when not in use. Store the unit in a clean and dry location.

CAUTION
DO NOT disassemble or attempt to repair this device or its components. This may cause an inaccurate reading.

CAUTION
Do not store the device or its components in locations that are exposed to vibrations or shocks.

Battery Replacement and Maintenance

- When the device shows that the battery is low, please replace the battery in time.
- If the device is not used for a long time (1 week or longer) remove the batteries and store them separately.

Calibration and Service

- The accuracy of this blood pressure monitor has been carefully tested and is designed for a long service life.
- It is generally recommended to have the unit inspected every two years to ensure correct functioning and accuracy. Please consult your authorised dealer or the Customer Service at the address given on the packaging or on included documents.

How to Enter the Test Mode

CAUTION
The Test Mode is mainly for professionals to enter the pressure calibration mode of the electronic sphygmomanometer and check the pressure value of the electronic sphygmomanometer through standard pressure meter test methods.

- Press and hold the  button  while installing the battery. Then the device enters the Test Mode, which is indicated on the display by showing “**CA**” and “**0**”.

Technical Specifications

Product description:	medisana Blood Pressure Monitor BW 315
Product Category:	Electronic Sphygmomanometer
Display:	LCD digital display
Cuff pressure range:	0 to 290 mmHg
Range of wrist circumferences:	12.5 to 21.5 cm
Blood pressure measurement range:	Systolic: 60 to 255 mmHg Diastolic: 30 to 199 mmHg Pressure: ±3 mmHg Pulse measurement range: 40 to 199 beats/minute Accuracy: ±5% of display reading
Static accuracy:	Phase I: systolic, Phase V: diastolic
Pulse:	Monitor: IP22 Automatic by electric pump Automatic pressure release valve Single automatic measurement
Measurement method:	 Type BF (wrist cuff) 3.0V DC 1.5W (direct current) 2 × 1.5V “AAA” alkaline batteries (LR03) approx. 300 measurements (using initially new alkaline batteries)
Durable period / service life:	5 years
Operating conditions:	5 to 40°C, 15% to 85% RH (non-condensing) 700 hPa to 1060 hPa
Storage & transportation condition:	−10 to 55°C, 15% to 95% RH (non-condensing) 500 hPa to 1060 hPa
Protection against electric shock:	internally powered
Dimensions:	approx. 70 ×64 ×31 mm
Weight:	105g (without batteries)
Memory:	2 × 120 measurements for 2 users (systolic, diastolic, pulse)
Article number:	51072
EAN Number:	40 15588 51072 4
Electromagnetic compatibility:	see the instruction manual that is part of the product delivery.

Disposal

 This marking, shown on the product or its associated documents, indicates that it should not be disposed of with other household waste. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this product from other types of waste and recycle it responsibly to promote the sustainable reuse of material resources. Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can return this item for environmentally safe recycling. Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial waste for disposal.

Warranty and repair terms

Your statutory warranty rights are not restricted by our guarantee below. Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **medisana** products is 3 years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.	
The current version of this instruction manual can be found under https://docs.medisana.com/51072 .	
Information about service can be found here: https://www.medisana.com/servicepartners	
	medisana GmbH , Carl-Schurz-Str. 2, 41460 NEUSS, GERMANY
	Shenzhen Combei Technology Co., Ltd. 11-5B, No.105, Huanguan South Road, Dahe Community, Guanhu Street, Longhua District, Shenzhen, 518110 Guangdong, P.R. China TEL: +86-755-29588956 EXT: 818, 809 http://www.combei.cn
EU REP	MedNet EC-REP GmbH , Borkstr. 10, 48163 Münster, GERMANY

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