|    |                                   |                                |
|----|-----------------------------------|--------------------------------|
| DE | <b>Blutdruckmessgerät</b>         | <b>Gebrauchsanweisung</b>      |
| GB | <b>Blood pressure monitor</b>     | <b>Instructions for use</b>    |
| NL | <b>Drukmetr</b>                   | <b>Gebruiksaanwijzing</b>      |
| FR | <b>Tensiomètre de bras</b>        | <b>Mode d'emploi</b>           |
| ES | <b>Tensiómetro de brazo</b>       | <b>Manual de instrucciones</b> |
| IT | <b>Sfigmomanometro da braccio</b> | <b>Istruzioni per l'uso</b>    |
| SE | <b>Blodtrycksmätare för armen</b> | <b>Bruksanvisning</b>          |
| PL | <b>Ciśnieniomierz naramienny</b>  | <b>Instrukcja obsługi</b>      |

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Alternatively, you can also operate the device with a **5V / 1A** mains powered USB-C cable. (Adapter and cable are not included in scope of delivery). Connect the cable to the socket . In this case the batteries may remain in the unit, they get mechanically disconnected. If the blood pressure monitor is no longer used, remove the cable from the blood pressure monitor and plug the mains adaptor out of the main's socket.

- The optional AC adapter should comply with the requirement of IEC60601-1:2005 + AMD1:2012 + AMD2:2020. In addition, all configurations shall comply with the requirements for medical electrical systems (see IEC 60601-1-1 or clause 16 of the IEC 60601-1, respectively). Anybody connecting additional equipment to medical electrical equipment configures a medical system and is therefore responsible that the system complies with the requirements for medical electrical systems. Please attention that local laws take priority over the above mentioned requirements. If in doubt, consult your local representative or the technical service department.
- This device is double insulated and protected against short circuit and overload by a primary thermal fuse. Equipment class 2.
- Insert the other side of the adaptor into the outlet with 100-240V.
- Adapter technical features: Input 100–240 V AC, 50/60 Hz. Output 5 V DC, 1 A

#### Settings

##### 1. User setting:

Press SET button  when power off.

The screen will display  or . Press MEM button to switch between  and . Press SET button  to confirm the user. Afterwards the computer will enter into user setting mode.

**2. Year setting:**

The input for setting the year then starts flashing. Keep pressing the MEM-button  until the required year appears. Press SET button  when you confirm the year, then it will switch to the month and date setting mode.

##### 3. Month and date setting:

The month starts flashing. Keep pressing the MEM-button  until the required month appears. Press SET button  to confirm the month, then computer will proceed to set the date. As with the month setting, keep pressing the MEM-button  until the required day appears. Press SET button  to confirm the date, then computer will switch to time setting mode.

##### 4. Time setting:

The hour starts flashing. Keep pressing the MEM-button  until the required hour appears. Press SET button  to confirm the hour and proceed in the same way for minute setting. Afterwards CL appears in the display. You can quit set-up mode by pressing SET button . If the batteries are changed the information is lost and the date and time need to be set again.

#### Fitting the cuff

- Push the end piece of the air tube  into the hole on the left side of the unit prior to use.
- While the open end of the cuff through the metal bracket so that the Velcro fastener is on the outside and it becomes a cylindrical form (Fig.1). Slide the cuff over your left upper arm.
- Position the air hose in the middle of your arm in line with your middle finger (Fig.2) (a). The lower edge of the cuff should be 2 - 3 cm above the crease of the elbow. (b). Pull the cuff tight and close the Velcro fastener. (c).
- Measure the pulse on your bare arm.
- Only position the cuff on the right arm if it cannot be used on the left arm. Always carry out measurements on the same arm.
- Correct measuring position for sitting (Fig.3).

#### Notes

- Only use the cuff provided by the manufacturer. Replacing cuff with other manufacturers parts will cause inaccurate measurement results.
- A loose or open cuff could cause false reading.
- With repeated measurements, blood accumulates in the arm which can lead to false reading. Consecutive blood pressure measurements should be repeated after 1 minute pause or after the arm has been held up in order to allow the accumulated blood to flow away.

#### Taking a blood pressure measurement

After the cuff has been appropriately positioned the measurement can begin:

- Switch the unit on by pressing the START/STOP button .
- If the START/STOP button is pressed, the display characters are shown (for a while). This is a display test, it can be used to check that the display is indicating all characters properly.
- The unit is ready for measurement and the number 0 flashes for 2 seconds. It automatically inflates the cuff slowly in order to measure your blood pressure. The cuff indicator  is shown on the display.
- The unit inflates the cuff until sufficient pressure has built up for a measurement. Then the unit slowly releases air from the cuff and carries out the measurement. When the device detects the signal, the heart symbol  on the display starts to flash.
- When the measurement is finished, the cuff is deflated completely. The systolic and diastolic blood pressure and the pulse value with the pulse symbol appear on the display . The blood pressure indicator flashes next to the relevant coloured bar depending on the blood pressure classification. If the unit has detected an irregular heartbeat, the symbol  also flashes.

**WARNING**

**Do not take any therapeutic measures on the basis of a self-measurement.**

**Never alter the dose of a medicine prescribed by a doctor.**

#### Self check of cuff strap

The cuff indicator  is shown on the display if the cuff is fastened correctly. If the cuff is too loose the symbol  is flashing. In this case, press he START/STOP button  to interrupt the measurement, then wrap the cuff again correctly and restart the measurement.

#### Movement indicator

The motion detector symbol  appears flashing on the display if the patient has moved during the measurement. Such motion can cause incorrect measurement values.

#### Discontinuing a measurement

If it is necessary to interrupt a blood pressure measurement for any reason (eg. the patient feels unwell) the START/STOP button  can be pressed at any time. The device immediately decreases the cuff pressure automatically.

#### Displaying stored values

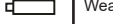
This unit features 2 separate memories, each with a capacity of 199 memory slots. Results are automatically stored in the memory selected by the user. Press the MEM button  when power off, to call up the measured values stored. The average of the last 3 measurements appears on the display together with the AVG symbol . Pressing the MEM button  again, the last measured value will be shown. Repeatedly pressing the MEM button  displays the respective values measured previously. If you have pressed a button, the unit automatically switches self off after approximately 60 seconds in memory recall mode. Memory recall mode can be exited at any time by pressing the START/STOP button , which will also switch the unit off. The oldest value is deleted if 199 measured values have been stored in the memory and a new value is saved.

#### Deleting stored values

If you are sure that you want to permanently remove all stored memories, press the SET button  for 7 times until CL appears when power off. Press the START/STOP button  to clear all the memories. After this press MEM button , M and "no" will be shown on the display which mean that no memory in store.

#### Error messages and error remedying

Error indicators: The following symbol will appear on the display when measuring abnormal.

| Symbol                                                                             | Cause                                                                                   | Remedying                                                                                                                                                    |
|------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------|
| E-1                                                                                | Weak signal or pressure change suddenly                                                 | Wrap the cuff properly. Remeasure with correct way.                                                                                                          |
| E-2                                                                                | External strong disturbance                                                             | When near cell phone or other high radiant device, the measurement will be failed. Keep quiet and no chatting when measure.                                  |
| E-3                                                                                | Inflation error                                                                         | Wrap the cuff properly. Make sure that the plug is properly inserted in the unit. Remeasure.                                                                 |
| E-5                                                                                | Abnormal blood pressure                                                                 | Repeat the measurement after relax for 30 mins. If get unusual readings for 3 times, please contact your doctor.                                             |
|  | Weak batteries                                                                          | The batteries are too low or empty. Replace all four batteries with new 1.5V type AA LR6 batteries.                                                          |
| E-U                                                                                | Problem with the cuff                                                                   | The inflation pressure fails to reach 30mmHg within 12 seconds. Wrap the cuff properly. Make sure that the plug is properly inserted in the unit. Remeasure. |
| Troubleshooting                                                                    |                                                                                         |                                                                                                                                                              |
| Problem                                                                            | Check                                                                                   | Cause and solutions                                                                                                                                          |
| No power                                                                           | Check the battery power. Check the polarity position.                                   | Replace batteries. Check for correct installation of the batteries (polarity).                                                                               |
| No inflation                                                                       | Check whether the plug is inserted correctly. Check whether the plug is broken or leak. | Insert into the air socket tightly. Change cuff.                                                                                                             |
| Err alarms & measurement is interrupted                                            | Movement or speaking during measurement.                                                | Do not move or speak during the measurement.                                                                                                                 |
| Cuff leaks                                                                         | Check whether the cuff is damaged or too loose.                                         | Wrap the cuff tightly. Change cuff.                                                                                                                          |

Please contact the customer service if you can't solve the problem. Do not disassemble the unit by yourself.

#### Cleaning and care

Remove the batteries before cleaning the unit. Clean the unit with a soft cloth lightly moistened with a mild soap solution. Never use abrasive cleaning agents, alcohol, naptha, thinner or gasoline etc.. Never immerse the unit or any component in water. Be cautious not to get any moisture in the main unit. Do not expose the unit to direct sunlight; protect it against dirt and moisture. Do not subject the unit to extreme hot or cold treatment. Keep the unit in the storage case when no use. Store the unit in a clean and dry location.

#### Cuff cleaning and disinfecting

Use a dust-free cloth dipped in water to wipe the cuff surface, until there is no visible stain and then use a dry dust-free cloth to wipe the water off the cuff surface. After cleaning, check the internal and external surfaces of the cuff to ensure that there are clean. If dirt is found, clean repeatedly until there is no visible stain and then place it in a cool, ventilated and dry place (not in direct sunlight as drying and drying is not allowed).

#### Disinfecting the cleaned and dried cuff

Wipe the cuff surface back and forth with a dust-free cloth dipped with 75% rubbing alcohol by volume to remove the sebum and other microorganisms from the surface for 1 minute each time, totally 10 times. After disinfection, dry it with a clean dust-free cloth to allow the residual alcohol to volatilize naturally. The cuff can be cycled and cleaned 10 times out of 1000 uses. When the cuff inflation time exceeds 20 seconds during normal measurement of the device and the device displays E-3, the cuff needs to be replaced. When the cuff bonding surface does not bind effectively, the cuff needs to be replaced. Please contact the manufacturer if you need to purchase a suitable cuff.

The cuff is consumable with a service life of 1,000 times (under company's test conditions). It is recommended to replace it in time to accurately measure blood pressure. If cuff is too small, please call the manufacturer or seller (retailer) for further information. In case of air leakage, contact the service centre for replacement. The cuff is treated as the applied part.

#### Guidelines / Standards

This Upper Arm Electronic Blood Pressure Monitor cannot be used in an MRI environment. It meets the requirements of the EU standard for noninvasive blood pressure monitors. It is certified in accordance with CE Guidelines and carries the CE symbol (conformity symbol) "CE"1123. The device meets the requirements of IEC 80601-2:20, Part 2 for non-invasive blood pressure monitors. Reporting adverse events for the users/patients/customers in EU countries: If users/patients/customers think that they or someone in the family has experienced a serious incident that has occurred in relation to the device, users/patients/customers are encouraged to report the incident to the manufacturer and the competent authority of the Member State in which the user/patient/customer is established.

#### Clinical Accuracy

The product was clinically investigated according to the requirements of ISO 81060-2 and complies with the accuracy requirements of +/-5mmHg at a standard deviation of max. 8mmHg.

#### Technical specifications

|                                                                          |                                                                                           |
|--------------------------------------------------------------------------|-------------------------------------------------------------------------------------------|
| Name and model                                                           | : Upper arm blood pressure monitor <b>BU 510</b>                                          |
| Display system                                                           | : Digital display                                                                         |
| Memory                                                                   | : 199 for measurement values                                                              |
| Measuring method                                                         | : Oscillometric                                                                           |
| Voltage supply                                                           | : 6V~4, x 1.5 V batteries AA LR6, or via USB-C powering 5V~4                              |
| Measuring range blood pressure                                           | : 0 – 295 mmHg                                                                            |
| Measuring range pulse                                                    | : 40 – 199 beats/min.                                                                     |
| Maximum measuring deviation of static pressure                           | : ± 3 mmHg                                                                                |
| Maximum measuring deviation of pulse values                              | : ≤ 5 % of the values                                                                     |
| Main unit lifetime                                                       | : 10,000 times under normal use                                                           |
| Battery life                                                             | : Could be used for 300 times at normal condition                                         |
| Pressure generation                                                      | : Automatic with pump                                                                     |
| Air release                                                              | : Automatic                                                                               |
| Autom. switching-off                                                     | : after approx. 1 minute                                                                  |
| Operating environment                                                    | : +5 °C to +40 °C, min. 15 to 90 % max. relative humidity, air pressure 70kPa to 106kPa   |
| Note: The product cannot be operated at an altitude of 2,000 m or above. |                                                                                           |
| Storage & transportation environment                                     | : -20 °C to +55 °C, min. 10 to 93 % max. relative humidity, air pressure 70kPa to 106kPa. |
| Avoid crash, sunlight or rain during transportation.                     |                                                                                           |

|                        |                                                                       |
|------------------------|-----------------------------------------------------------------------|
| Dimensions (L x W x H) | : approx. 130 x 111.5 x 56 mm                                         |
| Cuff                   | : 22 - 42 cm for adults                                               |
| Weight (main unit)     | : approx. 258 g zonder batteries                                      |
| Item number            | : 51160                                                               |
| EAN number             | : 40 15588 51160 8                                                    |
| Spare parts            | : Cuff 22 – 42 cm for adults<br>Art. No. 51161 / EAN 40 15588 51161 5 |

#### Disposal

This marking, shown on the product or its associated documents, indicates that it should not be disposed of with other household waste. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this product from other types of waste and recycle it responsibly to promote the sustainable reuse of material resources. Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can return this item for environmentally safe recycling. Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial waste for disposal.

#### Warranty and repair terms

Your statutory warranty rights are not restricted by our guarantee below. Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

- The following warranty terms apply:
  - The warranty period for **medisana** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
  - Defects in material or workmanship will be removed free of charge within the warranty period.
  - Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
  - The following is excluded under the warranty:
    - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
    - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
    - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
    - d. Applied parts which are subject to normal wear and tear.
  - Liability for direct or indirect consequential losses caused by a unit are excluded even if the damage to the unit is accepted as a warranty claim.

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.

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Electromagnetic compatibility information

Electromagnetic Compatibility – Guidance and Manufacturer’s Declaration

| Guidance and Manufacturer’s Declaration – Electromagnetic Emissions                                                                                                             |                |                                                                                                                                                                                                                                |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| This product is intended for use in the electromagnetic environment specified below. The customer or user of this product should ensure that it is used in such an environment. |                |                                                                                                                                                                                                                                |
| Emissions test                                                                                                                                                                  | Compliance     | Electromagnetic environment                                                                                                                                                                                                    |
| RF emissions CISPR11                                                                                                                                                            | Group 1        | The <b>BU 510</b> uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.                                         |
| RF emissions CISPR11                                                                                                                                                            | Class B        | The <b>BU 510</b> is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes. |
| Harmonic emissions IEC 61000-3-2                                                                                                                                                | Not applicable |                                                                                                                                                                                                                                |
| Voltage fluctuations / flicker emissions IEC 61000-3-3                                                                                                                          | Not applicable |                                                                                                                                                                                                                                |

| Guidance and manufacturer’s declaration – Electromagnetic Immunity                                                                                                              |                                                                                                                            |                                                                                                                            |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------|
| This product is intended for use in the electromagnetic environment specified below. The customer or user of this product should ensure that it is used in such an environment. |                                                                                                                            |                                                                                                                            |
| Immunity test                                                                                                                                                                   | IEC 60601 test level                                                                                                       | Compliance level                                                                                                           |
| Electrostatic discharge (ESD) IEC 61000-4-2                                                                                                                                     | ± 8 kV contact<br>± 2 kV, ± 4 kV,<br>± 8 kV, ± 15 kV air                                                                   | ± 8 kV contact<br>± 2 kV, ± 4 kV,<br>± 8 kV, ± 15 kV air                                                                   |
| Electrical fast transient/burst IEC 61000-4-4                                                                                                                                   | ±1 kV signal input/output<br>100 kHz repetition frequency                                                                  | ±1 kV signal input/output<br>100 kHz repetition frequency                                                                  |
| Surge IEC 61000-4-5                                                                                                                                                             | Not applicable                                                                                                             | Not applicable                                                                                                             |
| Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11                                                                             | Not applicable                                                                                                             | Not applicable                                                                                                             |
| Power frequency Magnetic field IEC 61000-4-8                                                                                                                                    | 30 A/m<br>50 Hz / 60 Hz                                                                                                    | 30 A/m<br>50 Hz / 60 Hz                                                                                                    |
| Conducted RF IEC 61000-4-6                                                                                                                                                      | 3V signal input/output;<br>0.15MHz–80MHz<br>6V in ISM and amateur radio bands between 0,15 MHz and 80 MHz<br>80% AM at 2Hz | 3V signal input/output;<br>0.15MHz–80MHz<br>6V in ISM and amateur radio bands between 0,15 MHz and 80 MHz<br>80% AM at 2Hz |
| Radiated RF IEC 61000-4-3                                                                                                                                                       | 10 V/m<br>80 MHz – 2,7 GHz<br>80% AM at 2 Hz                                                                               | 10 V/m<br>80 MHz – 2,7 GHz<br>80% AM at 2 Hz                                                                               |
| NOTE: U <sub>T</sub> is the a.c. mains voltage prior to application of the test level.                                                                                          |                                                                                                                            |                                                                                                                            |

| Guidance and manufacturer’s declaration – Electromagnetic Immunity                                        |                |                          |                                |                        |
|-----------------------------------------------------------------------------------------------------------|----------------|--------------------------|--------------------------------|------------------------|
| Immunity test                                                                                             | Test frequency | Modulation               | IEC 60601-1-2 test level (A/m) | Compliance level (A/m) |
| Radiated RF IEC 61000-4-39 (Test specifications for ENCLOSURE PORT IMMUNITY to proximity magnetic fields) | 30Hz           | CW                       | 8                              | 8                      |
|                                                                                                           | 134.2kHz       | Pulse modulation 2.1 kHz | 65                             | 65                     |
|                                                                                                           | 13.56MHz       | Pulse modulation 50kHz   | 7.5                            | 7.5                    |

**Statement:** “The **BU 510** of Arm Blood Pressure Monitor was tested according to the recommendations of Technical Report IEC TR 60601-4-2: Medical electrical equipment – Part 4-2: Guidance and interpretation – Electromagnetic immunity; performance of medical electrical equipment and medical electrical systems.”

Warning:

- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.”
- Don’t near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

**Notice:** If you think you or someone in your family has experienced a serious reaction to a medical product, please report to manufacturer and the competent authority of the Member State in which you are established.

| Guidance and manufacturer's declaration – electromagnetic emissions                                                                                                             |                      |             |                                                                |                                 |                |              |                                |                        |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------|-------------|----------------------------------------------------------------|---------------------------------|----------------|--------------|--------------------------------|------------------------|
| This product is intended for use in the electromagnetic environment specified below. The customer or user of this product should assure that it is used in such an environment. |                      |             |                                                                |                                 |                |              |                                |                        |
| Radiated RF<br>IEC 61000-4-3<br>(Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)                                                       | Test Frequency (MHz) | Band (MHz)  | Service                                                        | Modulation                      | Max. Power (W) | Distance (m) | IEC 60601-1-2 test level (V/m) | Compliance level (V/m) |
|                                                                                                                                                                                 | 385                  | 380 – 390   | TETRA 400                                                      | Pulse modulation 18 Hz          | 1.8            | 0.3          | 27                             | 27                     |
|                                                                                                                                                                                 | 450                  | 430 – 470   | GMRS 460, FRS 460                                              | FM ± 5 kHz deviation 1 kHz sine | 2              | 0.3          | 28                             | 28                     |
|                                                                                                                                                                                 | 710                  | 704 – 787   | LTE Band 13, 17                                                | Pulse modulation 217 Hz         | 0.2            | 0.3          | 9                              | 9                      |
|                                                                                                                                                                                 | 745                  |             |                                                                |                                 |                |              |                                |                        |
|                                                                                                                                                                                 | 780                  |             |                                                                |                                 |                |              |                                |                        |
|                                                                                                                                                                                 | 810                  | 800 – 960   | GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5         | Pulse modulation 18 Hz          | 2              | 0.3          | 28                             | 28                     |
|                                                                                                                                                                                 | 870                  |             |                                                                |                                 |                |              |                                |                        |
|                                                                                                                                                                                 | 930                  |             |                                                                |                                 |                |              |                                |                        |
|                                                                                                                                                                                 | 1720                 | 1700 – 1990 | GSM 1800, CDMA 1900, GSM 1900 DECT, LTE Band 1, 3, 4, 25, UMTS | Pulse modulation 217 Hz         | 2              | 0.3          | 28                             | 28                     |
|                                                                                                                                                                                 | 1845                 |             |                                                                |                                 |                |              |                                |                        |
|                                                                                                                                                                                 | 1970                 |             |                                                                |                                 |                |              |                                |                        |
|                                                                                                                                                                                 | 2450                 | 2400 – 2570 | Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7           | Pulse modulation 217 Hz         | 2              | 0.3          | 28                             | 28                     |
|                                                                                                                                                                                 | 5240                 | 5100 – 5800 | WLAN 802.11 a/n                                                | Pulse modulation 217 Hz         | 0.2            | 0.3          | 9                              | 9                      |
|                                                                                                                                                                                 | 5500                 |             |                                                                |                                 |                |              |                                |                        |
|                                                                                                                                                                                 | 5785                 |             |                                                                |                                 |                |              |                                |                        |

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Information about service can be found here:

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